

Pylon Crestal Surgical Guide Drilling Protocol

Implant Ø5.0

Guide Sleeve Offset 9.5

*Use proper bone condenser regarding the height.



Tissue Punch
Top Drill

Ø2.0	
12	16

Ø2.7/Ø3.2	
12	16

Ø3.2
17

Sinus Drill	
Ø3.2	
195	0.4 ~ 0.8cc

Bone Condenser

L 12

Final Drill		
Ø3.9	Ø4.3	Ø4.8
186	186	187

OFFSET
9.5